

lifetime limited warranty

Your Inspired by Drive branded product is warranted to be free of defects in materials and workmanship for the lifetime of the product for the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Inspired by Drive device or this warranty, please contact an authorized Inspired by Drive dealer.



2016 Medical Depot, Inc. All rights reserved.
Drive DeVilbiss Healthcare

Made in China



forearm crutch

item #FC100-2GB
#FC100-2GP
#FC100-2GR
#FC200-2GB
#FC200-2GP
#FC200-2GR
#FC300-2GB
#FC300-2GP
#FC300-2GR



EU Authorized Representative
Drive Medical LTD
Ainley's Industrial Estate
Elland, West Yorkshire,
United Kingdom HX5 9JP

adjusting leg height

1. Before use, adjust height of leg to your specific needs by depressing the push button and loosening the anti rattle collar (Figure 1).
2. When desired height is reached, lock leg into place by aligning button into proper hole, (Figure 2).
3. To be sure leg is locked into place by pushing down on or by pulling leg.

cleaning instructions

1. Use a non-abrasive mild detergent or a household cleaner with warm water.
2. Rinse with warm water and wipe dry.
3. **DO NOT USE AN ABRASIVE CLEANER OR CLOTH** with this product as damage may ensue.

FIGURE 1

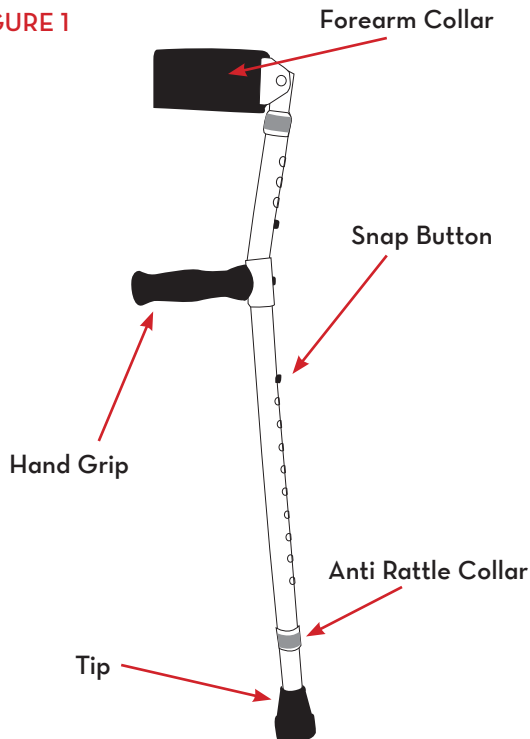


FIGURE 2

