

Etac Clean shower commode chairs



Etac Clean

Etac Clean is secure, safe and simple for both users and carers. With practical functionality and design, every detail of the chair has been developed to give the user a positive and safe bathroom experience.

Easy to fit and manouevre

Etac Clean is extremely easy to operate, thanks to the width of the push handle. The clever design can be seen throughout the entire chair. The upright handle ensures that Clean is a perfect fit, even over wall-mounted toilets, and the chair is open at the back to facilitate access.

Comfortable and practical foot support

The foot support is unique and can be pushed under the seat when not in use. The soft curved surface provides good comfort for the feet, which are also protected by the foot support's side protection.

All other variants of Clean also have these features, but with additional functionality to meet other needs.



Etac Clean shower commode chair		White	Lagoon gree
44 cm (171/4"), 4 lockable castors		80229269	8022920
49 cm (19¼"), 4 lockable castors		80229270	8022921
49 cm (19¼"), 2 lockable castors		80229273	8022921
49 cm (19¼"), 4 lockable wheels, backrest upholstery, panholder		80229286	
49 cm (19¼"), 2 lockable castors, soft back		80229305	
55 cm (21¾"), 4 lockable wheels, backrest upholstery, panholder		80229287	
55 cm (21¾"), 4 lockable castors		80229271	8022921
55 cm (21¾"), 2 lockable castors		80229274	8022921
55 cm (21¾"), 2 lockable castors, soft back		80229306	
60 cm (23½"), 2 lockable castors incl. heightening spacers		80229275	8022921
Etac Clean shower commode chair - double box		White	Lagoon gree
49 cm (19¼"), 4 lockable castors, double box		-	8022922
49 cm (19¼"), 2 lockable castors, double box		80229283	8022922
55 cm (21¾"), 4 lockable castors, double box		80229281	8022922
55 cm (21¾"), 2 lockable castors, double box		80229284	8022922
/eight	Material	Maintenance	
4 cm: 13.5 kg	Chassis: Powder-painted steel tubing	Clean the product with a solvent-free cleaning agent	
9 cm: 14.0 kg 5 cm: 14.5 kg	Seat/back support: Polypropylene Arm supports: Polypropylene	(pH level 5–9), or with a 70 % disinfectant solution. Can be decontaminated at max 85° C / 185° F if so	
) cm: 15 kg	Foot supports: Polypropylene	required.	
l aximum user weight 80 kg	Castor brackets: Polyamide Castors, lockable, ball-bearing: Polyamide/TPE		

Features



Slim but roomy The smart design gives Etac Clean a large and comfortable seating area even though the chair is neat and slim.




Easy access
The chair is open at the back
to facilitate access for
hygiene purposes.




Fits over the toilet
Thanks to the upright
operating handle, the chair is
a perfect fit, even over wallmounted toilets.



Foot support designed for feet The practical foot support has a soft, rounded shape which gives the feet extra comfort and protection.

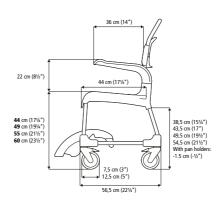

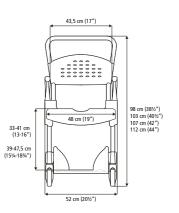



Comfort and support The soft enveloping se support provide stability, warmth and softness. Integrity in exposed situations Clean Comfort is designed to increase the feeling of integrity and dignity. The comfort padding protects and hides the user without reducing accessibility for the carers.


Etac Clean Comfort


All the features of Etac Clean can also be found in this chair, but with even greater comfort. The seat, arm support and back support are padded and guarantee soft and comfortable seating.

Soft and warmth for those who need it

Etac Clean Comfort enhances the feeling of well-being for those who need a little extra comfort and support in the bathroom. The water-repellent and heat-reflective cover of the back support and arm supports helps the user to stay warm. The contoured seat offers comfortable support for the thighs and envelops the user for a safer and more comfortable feeling.

Etac Clean Comfort has the same basic benefits as other members of the Clean family – but now with even greater comfort.

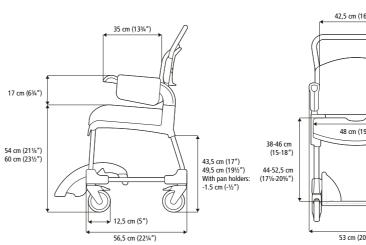

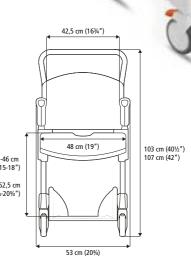

49 cm (19¼), incl. Comfort Cover, Comfort Seat Soft, 4 lockable castors, white 55 cm (21¾"), incl. Comfort Cover, Comfort Seat Soft, 4 lockable castors, white

80229298 80229299

Opening: depth x width 36.9 x 18.2 cm

Weight 49 cm: 13.5 kg 55 cm: 14 kg

Maximum user weight

Material

Comfort Cover: Polyethylene Comfort Seat Soft: Polyurethane Chassis: Powder-painted steel tubing Seat/back support: Polypropylene Arm supports: Polypropylene Foot support: Polypropylene Castor brackets: Polyamide
Castors, lockable,ball-bearing: Polyamide/TPE

Maintenance Clean the product with a solvent-free cleaning agent (pH level 5–9), or with a 70% disinfectant 185°F if so required.









The same chair for all heights The smart design means that the chair is a perfect fit above all wall-mounted toilets, regardless of height. Additional support for the legs In the event of additional need for leg and foot support, various accessories are available, e.g. split foot supports and calf supports.

Etac Clean Height Adjustable


In addition to all the features of the basic model, this variant allows you to adjust the height of the seat.


Adjust the height for a perfect fit

The possibility of adjusting the seat height means that Etac Clean Height Adjustable can be tailor-made to suit the user's needs – for example to make transfers easier or to fit perfectly over the toilet. Adjustment is simple and no tools are required.

Etac Clean Height Adjustable has the same basic benefits as the other members of the Clean family – but is also height-adjustable up to 60 cm, and has a greater range of foot supports and calf supports.



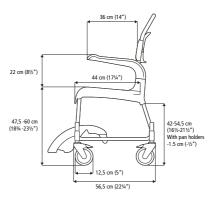

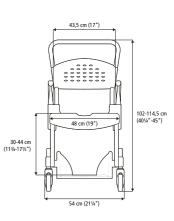

Etac Clean Height Adjustable

4 lockable wheels, white

14.2 ka

Maximum user weight

Material Chassis: Powder coated steel tubing Seat/Back support: Polypropylene Arm supports: Polypropylene Foot support: Polypropylene Castors with ball, bearings: Polyamide/TPE/ Stainless steel tubing
Castor height adjustment tubes: Stainless steel

Maintenance Clean the product with a solvent-free cleaning agent (pH level 5–9), or with a 70% disinfectant solution. Can be decontaminated at max 85°C / 185°F if so required.











Etac Clean 24"


This variant of Etac Clean is suitable for those who can operate the chair independently and want to maximise their mobility.


Easy to manoeuvre, for the independent user

The Etac Clean 24-inch wheels are positioned to make it easy to manoeuvre the chair without compromising user safety or comfort. It is easy to transfer to and from the chair thanks to the large, unrestricted seat in front of the wheels. The brake handles are also positioned below the seat so that they do not get in the way during transfer, while remaining easy to reach.

Etac Clean 24" has the same basic benefits as the other members of the Clean family – but allows the user to operate the chair independently.



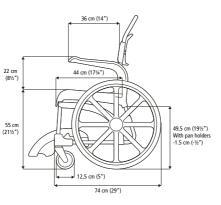

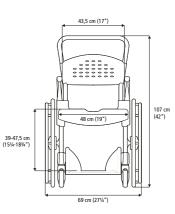

Etac Clean 24"

55 cm (21%) with 24" rear wheels, white 55 cm (21%) with 24" rear wheels, lagoon green

130 kg

Material Rear wheel adapter: aluminum Rear wheels: polyamide, solid tires Chassis: Powder-painted steel tubing Seat/back support: Polypropylene Arm supports: Polypropylene Foot support: Polypropylene Castor brackets: Polyamide Castors, ball-bearing: Polyamide/TPE

80229276 80229216

Maintenance Clean the product with a solvent-free cleaning agent (pH level 5–9), or with a 70 % disinfectant









Accessories



Soft back support 80209225 Adjustable. Washing at max 60°C (140°F).



Soft pad Grey: 80209260 Heat-reflecting with a non-slip surface. Recess width: 17 cm (6¾"). Material: polyethylene. Cleaning max max. 60° C (140 °F) or with alcohol wipe.



Side Support 80209509 Adjustable in width. Material: Polyester fabric, Velcro, polyethylene. Washing at max. 60°C (140°F). Can not be used in combination with: 80209225.



Soft back pad Grey: 80209261 Heat-reflecting with a non-slip surface. Material: polyethylene. Cleaning max. 60°C (140°F) or with alcohol wipe.



Crossbar 80209246 Attached to the arm supports. Material: Aluminium, cellular polyethylene, polypropylene. Can not be used in combination with: 80209228, 80209237 and 80209226.



Seat cover 80209247 When a complete seat is required. Material: Polyurethane. Cleaning max. 85°C (185°F) or with alcohol wipe.



Chest harness strap 80209442 Adjustable in length. Attach to soft back support. Material: Polyester.



Comfort Kit 80209237 Includes both Comfort Seat Soft and Comfort Cover. Cleaning max. 60° C (140 °F) or with alcohol wipe. Can not be used in combination with: 80209225 and 80209246.



Positioning belt 80209440 Adjustable in length. Attachment with velcro straps. Material: Polyester.



Comfort Cover 80209228 A soft back support with integrated arm support pads. Material: polyethylene. Cleaning max. 60° C (140 °F) or with alcohol wipe. Can not be used in combination with: 80209225 and 80209246.



Safety strap 80209034 Attachments on the rear side of the back support. Always use together with positioning belt. Material: Polyester. Can not be used in combination with: 80209225.



Comfort Arm Pads 80209226 Soft and warm, leaving the handles free to ensure a firm grip when standing up. Material: polyethylene. Cleaning max. 60° C (140 °F) or with alcohol wipe. Can not be used in combination with: 80209246.



Cushion, protect 80209245 Air-inflated ROHO cushion with excellent pressure distribution properties. Adjustable air pressure. Recess width: 21 cm (81/4"). Material: Neoprene. Cleaning max: 60°C (140°F).



Comfort Seat Soft Small opening width 15 cm (6"): 80209301 narrow opening width 18 cm (7"): 80209227 A soft contoured seat. Material: polyurethane. Cleaning max. 85° C (185°F) or with alcohol wipe.

Accessories



Soap dish (does not fit Clean 24") Grey: 80209266 Attaches to the side of the seat. Built-in shower handle holder. Material: Polypropylene. (Does not fit Clean 24")



Bucket holder Grev: 80209258 Fits Bucket with handle and lid and other buckets with a thick rim. Material: Polypropene.



Pan Grey: 80209254 Material: polypropylene. Pan incl. lid with handle

Material: Polypropylene.

Grey: 80209255



of pan/bucket Grey: 80209259 Fitted to the pan/bucket holder. Lowers the pan/bucket holder with 6 cm (2½"). Material: Polypropene.

Attachment for low mounting



Pan holder Grev: 80209257 Fits both pan, bedpan and regular hospital pans with a thin rim.



Disposable pans 80209282 Strong and firm. 50 pcs.



Bucket with handle and lid 80302026 The bucket holder is not included. Material: Polypropylene.



Disposable pan holder kit Grey: 80209229 Fits Disposable pan.

Additional accessories – Clean Height Adjustable only Clean 49 and 55 cm only




Flip up foot supports 80209250 Removable, adjustable height. Attachments included. Material: powder coated aluminum and polypropylene foot supports.



Material: ABS plastic, stainless.



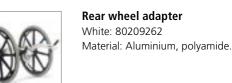

Additional accessories –



Heightening kit with wheels White, 4 pcs: 80209511 Lagoon green, 4 pcs: 80209512 Raise the seat by 5 cm (2").



75 mm (3") castors White, front, pair: 80209515 White, back pair: 80209516 Lagoon green, front, pair: 80209513 Lagoon green, back, pair: 80209514 Reduce seat height by 5 cm (2"). Mounted on Clean 44 cm as standard. Material: polyamide/TPE.

Etac Clean family 10 Creating Possibilities 11 Etac Clean family www.etac.com



"Everyone should be able to live a free and independent life and pursue their dreams regardless of any physical circumstances."

